

## HP PRO® SOLUTION CRANIAL BACK STOP

Article No. : 25000/6



### A. GENERAL PRODUCT INFORMATION

The products referred to in these instructions are medical devices, used for patient positioning and immobilisation in radiation therapy.

This product may only be used in combination with immobilisation masks produced by Orfit. Orfit prohibits the use of unauthorised third-party products in conjunction with its own products.

### B. PRODUCT DESCRIPTION

The Cranial Back Stop is an accessory of the ORFIT HP PRO SOLUTION. It is used in combination with the HP PRO base plate, special 3 and 5 point masks and a personalised head rest to form a reproducible patient positioning and immobilisation device.

Information on these other parts (and instructions on how to make the masks) can be found in the respective 'instructions for use' and on [www.orfit.com](http://www.orfit.com).

### C. USAGE

A cranial back stop is attached to the most cranial end of the base plate prior to moulding a thermoplastic mask. This assures a correct position of the head of the patient.

- Attach the cranial back stop to the HP PRO base plate



- Position the patient with the necessary accessories (e.g. head support) so the head touches the back stop



- Take away the cranial back stop
- Mould a thermoplastic mask

### D. PHYSICAL PROPERTIES



Dimensions : L 170 mm x W 85 mm x H 55 mm  
Weight : 0,1 kg  
Material : Polyamide (PA)

### E. STORAGE

Always store the product in a safe place to prevent it from getting damaged. Store the product between +10°C (50°F) and 40°C (105°F).

### F. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Do not attempt to make modifications or repairs yourself. Do not use the product when it is broken.

The product can be disposed of with household waste.

### G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website [www.orfit.com](http://www.orfit.com).

#### Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.  
HP PRO® is a registered trademarks of ORFIT Industries.



#### ORFIT INDUSTRIES

Vosveld 9A | B-2110 Wijnegem | Belgium  
T (+32) (0)3 326 20 26  
[welcome@orfit.com](mailto:welcome@orfit.com)

[www.orfit.com](http://www.orfit.com)



Ref. No. 50114

VERSION 4

LAST UPDATE: 21/03/2023

REVISION DATE: 21/03/2025