


HIGH PRECISION BASE PLATE-TO-COUCH TOP FIXATION DEVICES: TABLE FIX

Article No. : 35744/6

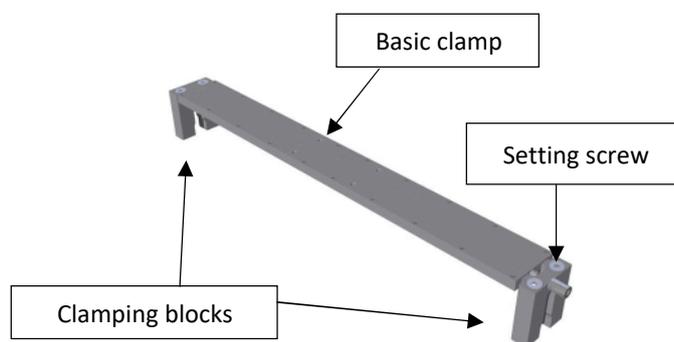

A. GENERAL PRODUCT INFORMATION

This product is a fixation device to attach High Precision base plates to non-indexed couch tops.

B. PRODUCT DESCRIPTION

Fixation system with clamping mechanism for use on non-indexed couch tops.

C. FIXATION TO BASE PLATES

Fixation to the 32150, 32201, 32204 and 32301 base plates




- Remove the clamping block on the side without the turning knob.
- Slide the Table fix device into the tunnel of the base plate until centred.
- Use the four M5 x 10 countersunk screws to fix the Table Fix to the base plate (use the 35744/12 M5 x25 in case this has to be done on a 32204 base plate).
- Refit the clamping block.
- Adjust the width of the Table Fix to the width of the couch by means of the setting screw.
- Be careful not to tighten too much to prevent damage to the internal mechanism.

Remarks:

- See next paragraph on how to mount and remove the clamping blocks.
- Use the four M5 x 20 with washers in case the Table Fix is mounted on a first-generation base plate with top fixation holes.


Fixation to 32110, 32130, 32111, 32090, 32095, 35751N and 35754/6N base plates

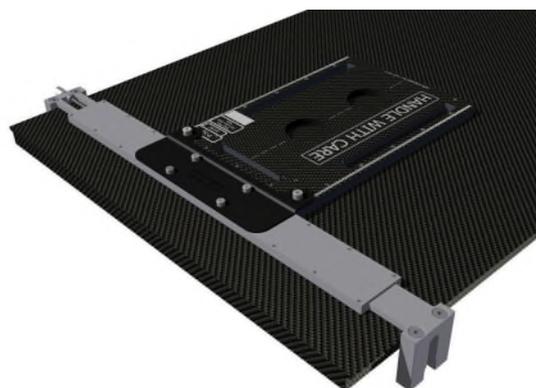

- The above base plates are connected with the Table Fix by means of a 35747 Adapter Plate.
- Attach the Adapter Plate to the Table Fix with the four M5 x 10 screws.
- Attach the base plates to the Adapter Plate with the two M6 x 10 screws.

D. MOUNTING OF THE CLAMPING BLOCKS

Different clamping blocks (long, small, narrow, L-shape, etc.) can be mounted on the Table Fix device to allow the fixation of the base plates to any type of couch top. These clamping blocks can be mounted on the Table Fix with the two M6 countersunk screws. Use the appropriate Allan Wrench to fasten or release these screws.

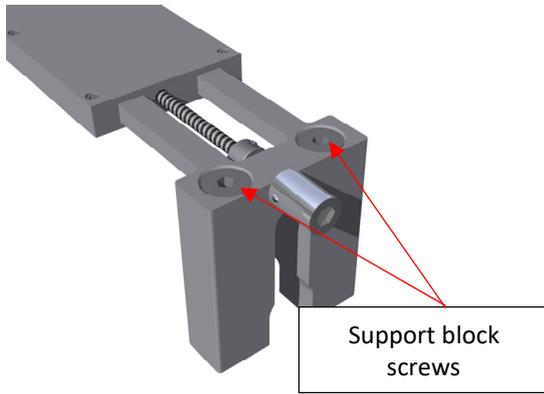



E. AVAILABLE TYPES OF CLAMPING BLOCKS

The following optional clamping blocks can be mounted to the Table Fix (see right).

(the top dimensions of the blocks are always 50 x 15 mm)

	<p>Art. No. 35745/11 Table Fix Block Standard Suitable for Table Tops with unlimited free vertical space along the side</p> <p><i>Max. table width: 590 mm (23.23") Min. table width: 460 mm (18.11")</i></p>
	<p>Art. No. 35745/9 Table Fix Block Short Suitable for Table Tops with limited free vertical space along the side (like CT-scanner couches, etc.)</p> <p><i>Max. table width: 590 mm (23.23") Min. table width: 460 mm (18.11")</i></p>

	<p>Art. No. 35745/12 Table Fix Block Standard L-Shape Suitable for narrow Table Tops</p> <p><i>Max. table width: 550 mm (21.65") Min. table width: 410 mm (16.14")</i></p>
	<p>Art. No. 35745/13 Table Fix Block Short/Small Suitable for wide Table Tops with limited free vertical space along the side (when the Table Fix should fit between the couch and a rail)</p> <p><i>Max. table width: 605 mm (23.82") Min. table width: 475 mm (18.7")</i></p>
	<p>Art. No. 35745/14 Table Fix Block Short L-shape Suitable for narrow Table Tops with limited free vertical space along the side</p> <p><i>Max. table width: 550 mm (21.65") Min. table width: 410 mm (16.14")</i></p>
	<p>Art. No. 35745/15 Table Fix Block Long L-shape Suitable for narrow Table Tops with limited free vertical space along the side</p> <p><i>Max. table width: 530 mm (20.87") Min. table width: 390 mm (15.35")</i></p>

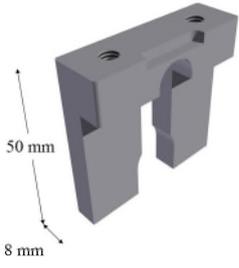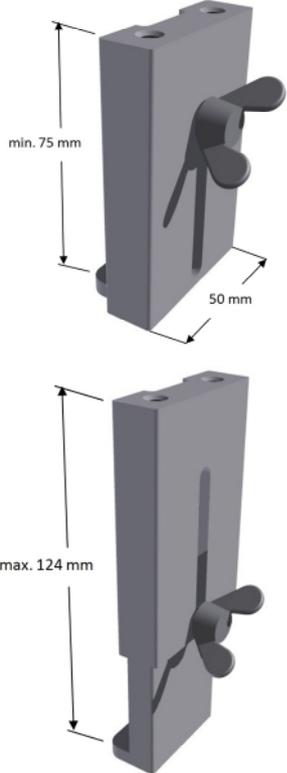	<p>Art. No. 35745/16 Table Fix Block Standard Small Suitable for wide Table Tops with large rounded edges and limited space on the sides due to a rail</p> <p><i>Max. table width: 605 mm (23.82")</i> <i>Min. table width: 475 mm (18.7")</i></p>
	<p>Art. No. 35745/17 Adjustable Table Fix Block Suitable to clamp down base plates on the rails of a couch</p> <p><i>Max. table width: 605 mm (23.82")</i> <i>Min. table width: 475 mm (18.7")</i></p>

F. PRECAUTIONS

Periodic checks of the products should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself**, but contact your distributor if there are any questions or concerns.

G. STORAGE

Always store the product in a safe place to prevent it from getting damaged.

H. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns.

The product can be disposed of with household waste.

I. FIXATION DEVICES COMBINATION PACKAGES

This product is part of following fixation devices combination packages:

- 35745
- 33124
- 33144
- 33145
- 33136
- 32158

J. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.



ORFIT INDUSTRIES
Vosveld 9A | B-2110 Wijnegem | Belgium
T (+32) (0)3 326 20 26
welcome@orfit.com

www.orfit.com



Ref. No. 31815
VERSION 8
LAST UPDATE: 21/03/2023
REVISION DATE: 21/03/2025