

HOSE CONNECTOR FOR HOSE ID4



Article No. : **18097**

A. GENERAL PRODUCT INFORMATION

The product referred to in these instructions is an adapter to connect Orfit Vacuum Pumps with Orfit Vacuum bags used for patient positioning and immobilisation in radiation therapy.

This product may only be used in combination with Orfit immobilisation products. Orfit prohibits the use of unauthorised third-party products in conjunction with its own products.

B. PRODUCT DESCRIPTION

This hose connector is used to connect Orfit Vacuum Pumps (ref. 18058 & 18076) with the Orfit Orange Vacuum Bags. The hose connector is delivered with the Orfit Vacuum Pumps.

The thinnest part of the hose connector is attached to the hose that is delivered with the Orfit Vacuum pumps as shown in the picture below.



The thickest part of the hose connector is attached to the tube of the Orfit Orange Vacuum Bags.



Information on the other parts can be found in the respective 'instructions for use' and on www.orfit.com.

C. STORAGE

Always store the product in a safe place to prevent it from getting damaged.
Store the device between +10°C (50°F) and 40°C (122°F).

D. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns.

The product can be disposed of with household waste.

E. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.

