

RAYCAST® HIGH PRECISION BASE PLATE-TO-COUCH TOP FIXATION DEVICES HIGH PRECISION BASE PLATE-TO- COUCH TOP FIXATION DEVICES TWO-PIN BAR – Set with Pins

Article No. : 32165/6



A. GENERAL PRODUCT INFORMATION

This product consists of parts for fixation devices which attach Raycast® High Precision base plates to couch tops with the 2-pin bar type indexing. This product may only be used in combination with Orfit immobilisation products. Orfit prohibits the use of unauthorised third-party products in conjunction with its own products.

Parts list:


2x	32165/1	Pin For 2-Pin Bar
2x	32165/11	Screw M5 x 10

B. PRODUCT DESCRIPTION

This set consists of two high precision white pins with required screws to replace older type, usually black, pins on every type of 2-pin bar. These new white pins are used to reduce play on the existing 2-pin bar.

C. HOW TO INSTALL

Remove the existing pins of your 2-pin bar and mount the new

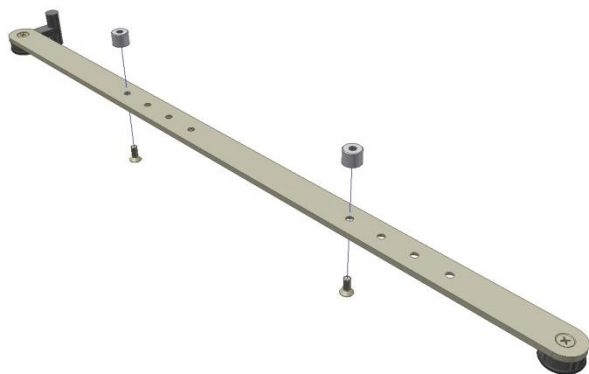

white pins on the same position using the screws included in the package.

D. STORAGE

Always store the product in a safe place to prevent it from getting damaged.

Store the product between +10°C (50°F) and 40°C (122°F).

E. PROPERTIES

Physical Properties

Dimensions pin : D 12,6 mm x H 9,5 mm

Weight: < 0.01 kg

This product is made completely of plastics.

This product is made entirely of electrically non-conductive, non-metallic and non-magnetic materials and is MRI safe.



F. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of soapy water or an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Contact your distributor if there are any questions or concerns.

The product can be disposed of with household waste.

G. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website www.orfit.com.

Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries. RAYCAST® is a registered trademarks of ORFIT Industries.



ORFIT INDUSTRIES
Vosveld 9A | B-2110 Wijnegem | Belgium
T (+32) (0)3 326 20 26
welcome@orfit.com

www.orfit.com



Ref. No. 50019
VERSION 3
LAST UPDATE: 02/04/2021
REVISION DATE: 02/04/2023